INSTRUCTIONS FOR USE

OneSett™

Subcutaneous Administration Set

EN	OneSett™ Subcutaneous Administration Set	- Instructions for use (page 2)
DA	OneSett™ Sæt til subkutan administration	- Brugsanvisning (side 4)
FI	OneSett™ Subkutaaninen annostelusarja	- Käyttöohjeet (sivu 6)
FR	OneSett™ Kit d'administration sous-cutanée	- Model d'emploi (page 8)
NO	OneSett™ Subkutant administreringssett	- Bruksanvisning (side 10)
SV	OneSett™ Subkutant administreringsset	- Bruksanvisning (sidan 12)
TR	OneSett™ Subkütan Uygulama Seti	- Kullanım talimatı (sayfa 14)

Consult
Instructions
for use

Caution

Sterilized
Using
Radiation

Do not
Reuse

Prescription
Only

Do Not
Resterilize

Non-
pyrogenic

Do not use if
package is
damaged

Unique Device Identifier

Medical Device

Store at room temperature.

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33-Suite 123
2595 AM The Hague
The Netherlands

UKRP
MedEnvoy UK Limited
85, Great Portland Street
First Floor London, W1W 7LT
United Kingdom

The OneSett™ Subcutaneous Administration Sets are a trademark of Innovative Health Sciences LLC (IHS). IHS is ISO 13485 certified and abides by the quality policies as outlined in 21 CFR 820.

© 2025 Innovative Health Sciences LLC. All Rights Reserved.

INNOVATIVE HEALTH SCIENCES LLC

1108 Kings Highway, Suite #4 Chester, NY 10918 USA
+1-855-680-0630 | www.innohealthsci.com

Page 1 of 16

EN INSTRUCTIONS FOR USE *Quick Start Guide*

OneSett™

Subcutaneous Administration Set

Indications for Using the OneSett™

The OneSett™ is intended for use with the Insignis™ Syringe Infusion System in the subcutaneous administration of 13-17cP 20% immunoglobulin solutions with infusion systems operating with a safe 13.5psi constant pressure. Consult the drug manufacturer's package insert for recommendations.

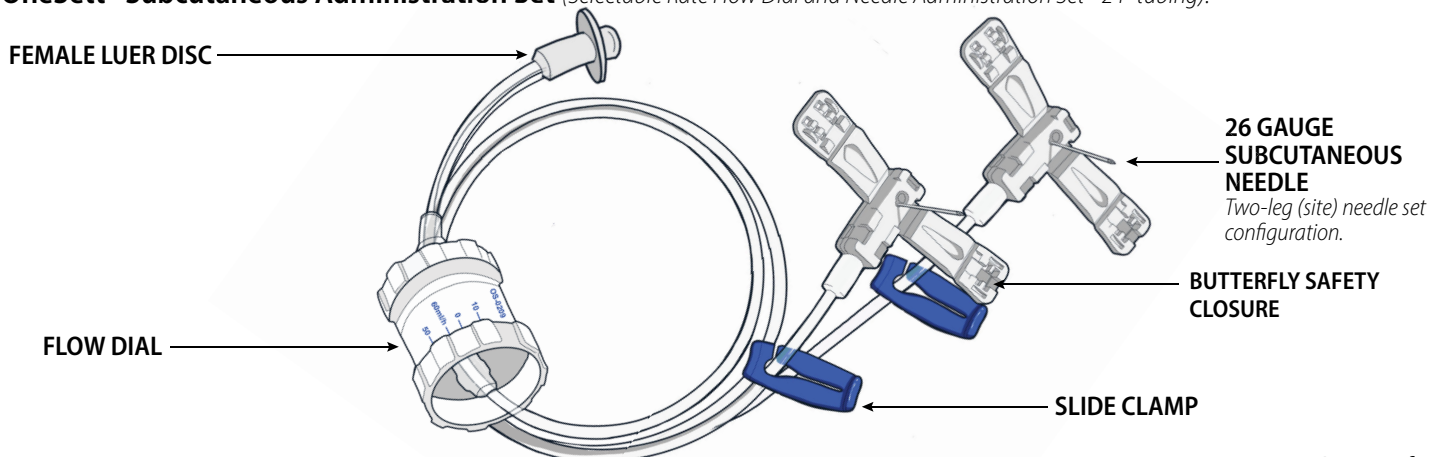
Precautions and Warnings

- The administration sets are single-use only. Do not re-sterilize.
- IHS advises against off-label use with the Insignis™ Syringe Infusion System.
- Never prime the infusion set or attempt to free a clogged tubing line while the set is inserted.
- Do not stretch or kink the delivery sets.
- It is recommended to store the administration sets at room temperature (16-30°C; 61-86°F).
- Failure to store in correct storage conditions may lead to possible leak at the controller.

Caution

- Use of the OneSett™ is intended only for the patient for whom the device is prescribed.
- If using the infusion set for the first time, do so in the presence of a healthcare provider.
- Use recommended Becton Dickinson and Company® 50ml #300865 or similar syringes with the OneSett™.
- Before use, carefully inspect the administration set package. Do not use if the package has been opened or is damaged.
- Always follow the infusion instructions located on the package insert of the manufactured drug.
- Verify the drug volume in the syringe(s) prior to beginning the infusion.
- Store infusion sets in a cool, dry place.
- Do not leave infusion sets in direct sunlight or inside a vehicle.
- If for any reason you experience a medical emergency during use, call your local emergency number and/or your healthcare provider immediately.
- Use of flow rate calculation software and/or computation documentation (tables, charts, graphs, etc.) may be unnecessary for use with the OneSett™ as the flow dial indicates the exact flow rate delivered to the patient.
- Always consult and comply with guidance provided by your healthcare providers and drug manufacturers regarding the location and the number of infusion sites.
- Each OneSett is calibrated from 10-60ml/hr/site for the number of needles included. Each needle site can deliver up to 60ml/hr of 13-17cP 20% immunoglobulin solution. A four-site needle configuration can provide a total flow rate of 240ml/hr (60ml/hr/site).
- If a needle site is clamped-off while using a multi-needle set, the flow rate to the remaining needle site(s) will increase. To compensate for this increase, it is advised to manually decrease the flow rate on the flow dial. See flow rate compensation recommendations on the next page.
- When using a two, three, or four needle set configuration, over-infusion may occur to remaining sites if one or more of the needles is blocked.
- Inaccurate medication delivery, infection, and/or site irritation may result from improper needle insertion and maintenance of the infusion site.
- Ensure that the needle is at 90°.
- Be sure to remove the needle guard prior to insertion.
- Since individuals inherently have varying amounts of subcutaneous fat on the body, be sure to select the appropriate needle length to fit your needs.
- For suspected site reactions, perform subsequent infusions in alternative sites.
- To stop the flow immediately, use the slide clamp and/or set the OneSett™ dial to 0ml/hr.
- Ensure the syringe is empty before removing the syringe from the driver. If medication is still in the syringe, activate the lever once to infuse the remaining medication.

OneSett™ Subcutaneous Administration Set *(Selectable Rate Flow Dial and Needle Administration Set - 24" tubing):*

EN INSTRUCTIONS FOR USE *Quick Start Guide*

Starting the Infusion for Subcutaneous Administration using the OneSett™:

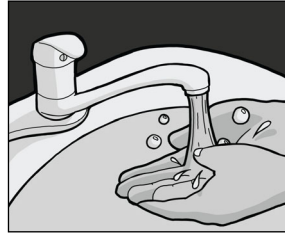
1 Ensure you have all supplies for your infusion.

Verify that you are using the correct administration set, making sure to check the number of sites and needle length on the packaging.

2 Wash hands

Wash hands thoroughly and if needed, wear disposable gloves.


3 Fill Syringe

Make sure the drug product is at room temperature and fill one or more 50ml BD® or similar syringe(s) to your required dose. Note: it may take several syringes to receive your prescribed dose.


4 Attach OneSett™

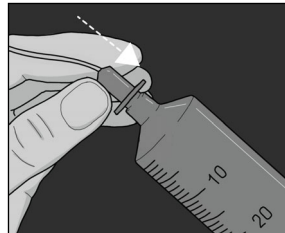Using aseptic technique, remove the end cap on OneSett™ and attach to the luer on 50ml BD® or similar syringe; twist to lock.


5 Prime OneSett™

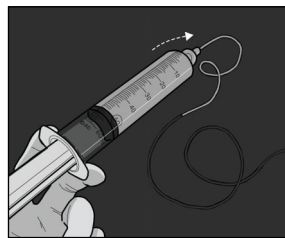Set OneSett™ to 60ml/hr. Gently press on syringe plunger and prime set, taking care to stop before medication reaches the needles. Set OneSett™ to 0ml/hr (OFF).


6 Prep Infusion Site(s)

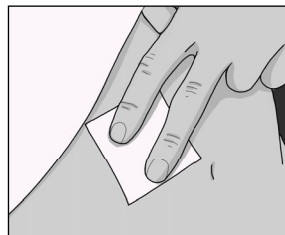Using aseptic technique, prep infusion site as directed by clinician. Remove rubber band from butterfly, fold wings back, and remove needle guard.


7 Insert Needle(s)

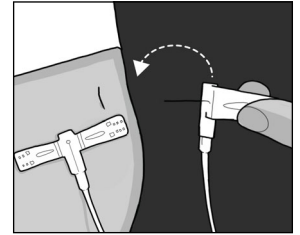Insert needle(s) into appropriate locations in subcutaneous tissue at 90° angle. Lay butterfly wings flat on skin. Cover site with tape or adhesive dressing.


8 Activate Infusion Pump

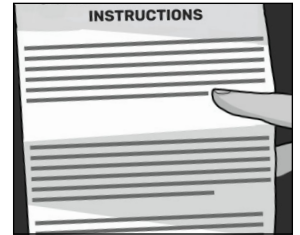Refer to the infusion pump instructions for use provided by the manufacturer to load and activate the infusion pump.

9 Set Flow Rate


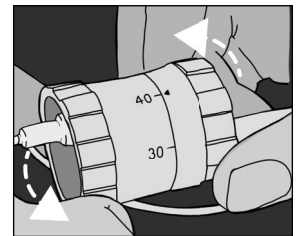
As advised by your healthcare provider, set appropriate flow rate and begin administration. Do not exceed the maximum flow rate as indicated by your provider.

Ending the Infusion:

1 Remove Dressing

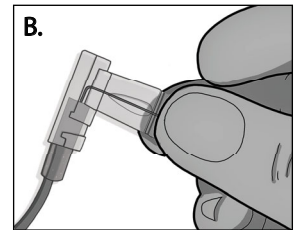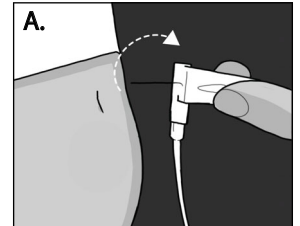Pause before removing needles; allow site pressures to decrease naturally. Remove needle dressing. To minimize accidental risk of needle stick injury, use one hand to remove needle. Using the same hand, squeeze butterfly safety closure wings together to firmly encase needle.

2 Dispose of Materials

Carefully dispose of materials in a sharps container per local regulations.


Product Identifiers

OneSett™ Subcutaneous Administration Set (box of 10)

Configuration	6mm	9mm	12mm	14mm
1 site	OS-0106	OS-0109	OS-0112	OS-0114
2 sites	OS-0206	OS-0209	OS-0212	OS-0214
3 sites	OS-0306	OS-0309	OS-0312	OS-0314
4 sites	OS-0406	OS-0409	OS-0412	OS-0414

Sterile Dressing

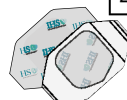Sterile dressing/adhesive is included in the administration set box.

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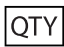
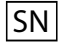
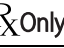
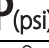
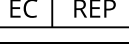
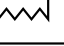
Subcutaneous Administration Set

Definition of Symbols

	Caution		Quantity		Do not reuse
	Consult Instructions for Use		Serial number		Catalog number
	Use by YYYY--MM-DD		Prescription only		Batch code
	Manufacturer		Sterilized using radiation		Volume
	Temperature		Humidity		Altitude
	Pressure device		Non-pyrogenic		European Conformity
	Pinch point		European Authorized Representative		Do not use if package is damaged
	Do not use in MRI setting		Do Not Resterilize		Medical Device
	Unique Device Identifier		Date of Manufacture		

CH REP
MedEnvoy Switzerland
 Gotthardstrasse 28
 6302 Zug
 Switzerland

EC REP
MedEnvoy Global B.V.
 Prinses Margrietplantsoen 33-Suite 123
 2595 AM The Hague
 The Netherlands

UKRP
MedEnvoy UK Limited
 85, Great Portland Street
 First Floor London, W1W 7LT
 United Kingdom

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INNOVATIVE HEALTH SCIENCES LLC
 1108 Kings Highway, Suite #4 Chester, NY 10918 USA
 +1-855-680-0630 | www.innohealthsci.com
 04-1038v4